// RF-MED

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Destination and use

WARNING

Improper use. Ignoring this message may lead to **severe injuries or death or to material damage!** Use the foot switch only in accordance with the information specified on the data sheet.

NOTICE

Improper use. Ignoring this message may lead to **material damage!** Never use the connecting cable to move or transport the foot switch. The connecting cable may come loose, break, or the isolation may be damaged. Use the carrying handle or take the housing to move the foot switch cautiously.

The foot switches of series MED serve as actuator for medical devices. They control medical appliances together with the control unit downstreamed by the marketer. The devices are provided with reed contacts or microswitches and are suitable for applications from 1 mA to approx. 5 A. This depends on the utilized switch insert. Always observe the information given in the data sheet of the foot switch! Housing and pedal are made of aluminium die-casting. The connecting cable is firmly fixed to the housing by buckling resistant and traction relief. If the switch is suitable for AP, there is a corresponding label on the switch. The actuating elements must only be used with a max. force of 1350 N. If a greater force is used, single functions and the whole device may fail.

Mounting / Wiring

Place the foot switch on a safe, even surface. Wire the foot switch according to the specified wire colours/terminal labelling. Contact symbols are shown for a not actuated switch. The cables are connected to the enclosure by cable glands with traction relief. Check plug connections for tight fit. Do not apply any mechanical pressure.

Notices

Read and follow the instructions given in these instructions for use. steute assumes no liability for damage resulting from ignoring these instructions for use, mishandling, improper application or unauthorized work on the foot switch. Do not use the foot switch in explosive areas. The electrical connection may only be carried out by authorised personnel. The application of a foot switch bearing the CE mark corresponding to the Medical Devices Act does not exempt the marketer from proving again the conformity as per Medical Devices Act (MDD 93/42/EEC) of the entire system! Despite careful production and inspection as well as redundant design of important components, an entirely safe function at any time cannot be guaranteed (in case of error a switching insert e. g. cannot switch off properly or a spring may break). Mishandling may lead to actuating elements causing injuries

such as squashing. The remaining risks have to be taken into consideration concerning the evaluation of the entire system! Additionally, magnetically conducting materials are to be kept away from the direct foot switch area at all times! AP models' connection to end products must be protected against accidental disconnection or connection, and disconnection shall only be performed with a tool. AP models connected to the end product must be protected from electrostatic charge. In case of application of a shielded control cable, the shield may only be used as functional earth conductor! Technical changes reserved.

Maintenance / Cleaning

WARNING

Unintentional actuation. Ignoring this message may lead to **severe injuries or death!** Never carry out maintenance work or functional testing during patient-related or medically relevant applications. Always check functions before restarting the foot switch.

In consideration of the following indications, the steute MED foot switches need only minor maintenance. However, according to ambient conditions and application frequency, we recommend a regular check or maintenance as follows:

1. Function test, check actuating elements for proper work.
2. Check housing and connecting cable for damage and harmful dirt.
 - When cleaning the device, use only water and a damp cloth rag impregnated with a mild cleanser.
 - Do not use cleaning agents which may damage the surfaces, such as detergents, abrasive cleansers or solvent-based cleaners (such as benzine, stain remover). Observe the cleaning agent manufacturer's instructions!
 - Only clean manually.

Disposal

- Observe national, local and legal regulations concerning disposal.
- Recycle each material separately.

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English

Technical data

Applied standards	EN 60601-1; EN 60601-1-2; EN 60601-2-2 ¹⁾ ; EN 60601-2-22 ²⁾ ; EN 60601-2-43 ¹⁾ ; EN 60529 / IEC 60529:1989 + A1:2000
Class	class I as per Medical Device Act / MDD 93/42/EEC
Enclosure	enamelled or powder-coated aluminium die-cast
Switching system	reed contact: 1S, 2S micro switch: 1PW, 2PW
Degree of protection ¹⁾	IP X1 to IP X8 according to IEC 60529
Connection	firmly attached control line, certified according to UL-Style-No. 2464/1061
Switching voltage	max. 25 VAC / 60 VDC
Switching current	max. 1 A
Switching capacity	reed contact: max. 30 VA
Mechanical life	> 1 million operations
Degree of pollution	2 (IP X6 to IP X8) and 3 (IP X1 to IP X5)
Storage temperatures	-20 °C to +70°C
Ambient temperature	-10 °C to +60°C
Relative humidity	10 - 100 %
Air pressure	storage: 500 - 1060 hPa environment: 800 - 1060 hPa
Altitude	max. 2000 m

- 1) EN 60601-2-2 and EN 60601-2-43 are only applied to products that have at least a IP X7 degree of protection. Foot controls with IP X1 to IP X5 degree of protection must not be used in areas where the occurrence of fluids at PROPER USE is most likely!
- 2) When EN 60601-2-22 is applied to products that are not protected by special safety equipment (bar, protective housing, shroud ...) against unintentional actuation, the persons responsible for placing on the market have to take care of required safety precautions.

Zusatzinformation zu Montage- und Anschlussanleitungen Additional information on mounting and wiring instructions Information complémentaire aux instructions de montage et de câblage Ulteriori informazioni sulle istruzioni di collegamento e montaggio Informação adicional para as instruções de montagem Дополнительная информация по монтажу и инструкциям по подключению

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This mounting and wiring instruction is also available in your national language on request.

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